


Version 6.0A compatible with v2-4.0 UC-102 version or higher

Nurse Call System **udeCOM**

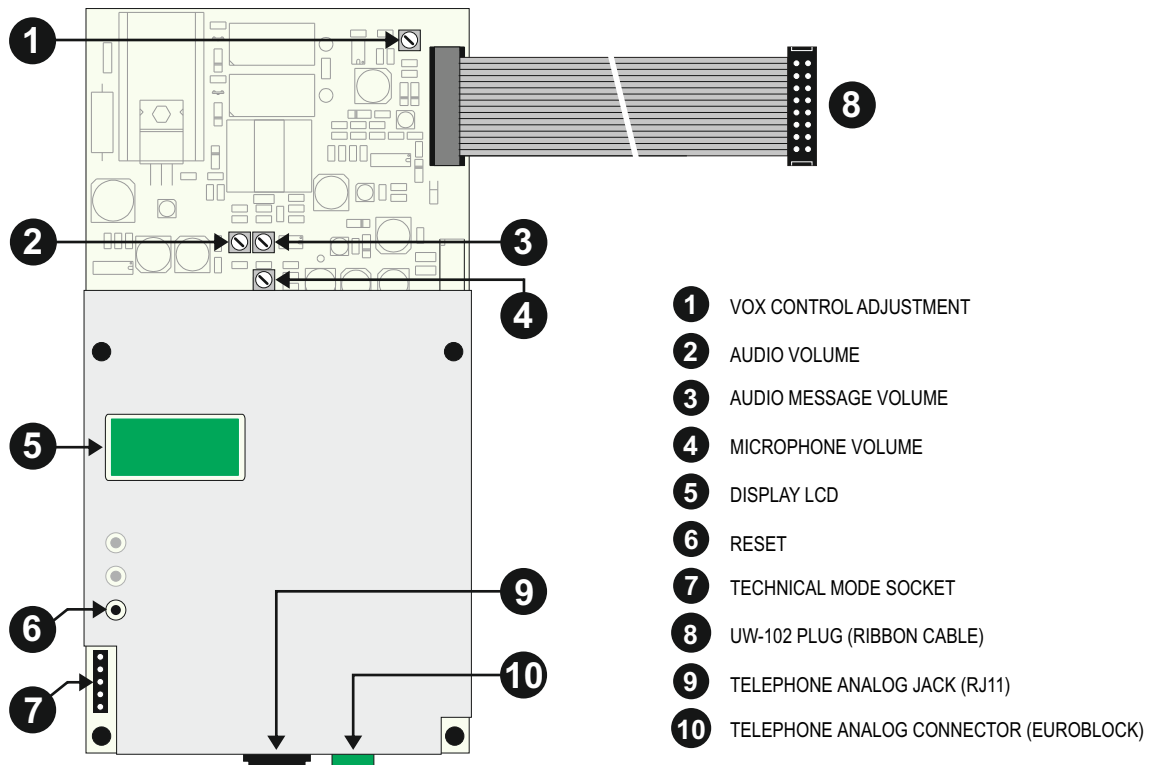

### CHARACTERISTICS

The UW-18 allows the intercom audio connection between UC-104 (room substation) and a telephone. The system will provide the bedroom number, floor and type of call through an audio message. This interface can be connected to a PABX or just a wireless analog station phone.

### CONFIGURATION:

#### Plug and play

Once installed the phone interface (UW-18) and plugged to the UW-102 it will work completely automatic.

#### Phone connection

- Insert the plug of the telephone cord into the jack input (click into the place) or connect to the euroblock terminal.
- Place the cord in cable duct.
- Insert telephone jack into telephone socket.

#### Audio adjustments:

The UW-18 has an audio factory configuration. You can adjust the intercommunication volumes (phone loudspeaker and microphone) and the system audio message volume.

1. Choose the potentiometer you want to adjust
2. Tune the required level.


#### Display messages

Anytime the display shows the operating state of the UW-18 like attending calls, playing audio message, etc. This information is only for technical staff and can give information of the system.

If you connect the associated telephone interface to a digital central station, an analog-digital adapter is mandatory.



### Central Station UC-102

Assembly the telephone interface UW-18 inside the UW-102 module and connect it to the connector UW-18 number 1 of the main board.

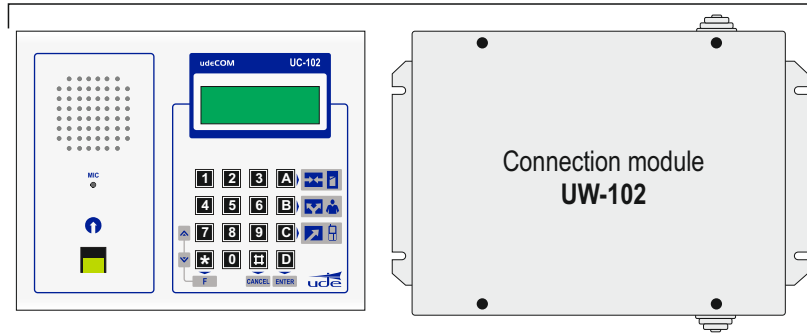

### ASSEMBLY AND WIRING

- 1 Open the UW-102 connection module
- 2 Unscrew and remove the mechanical surface mount
- 3 Fix the UW-18 in the left position

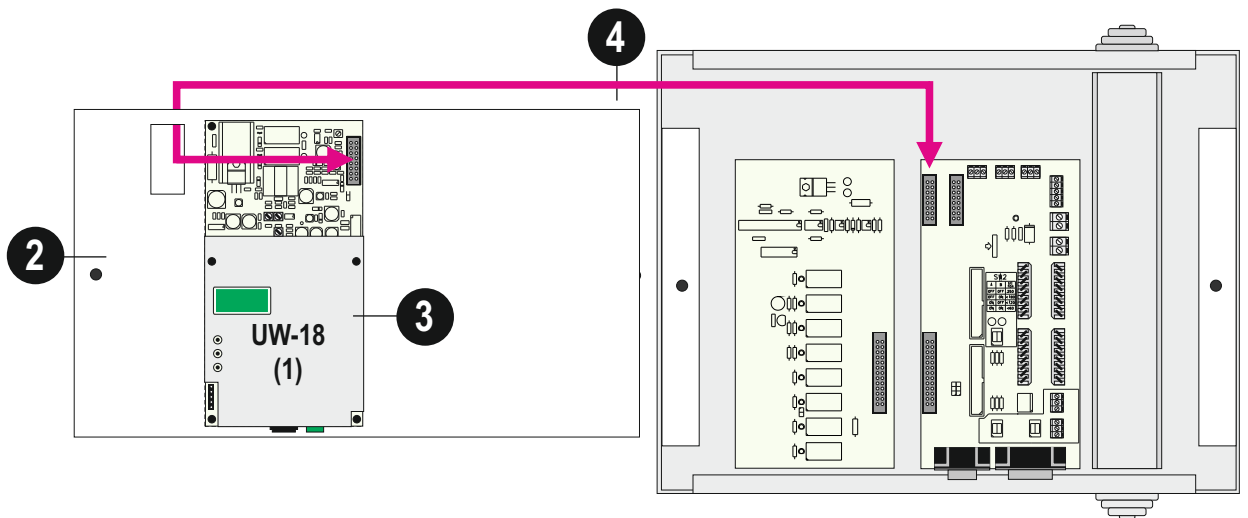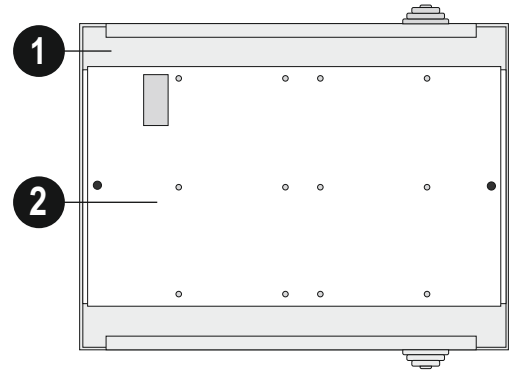

- 4 Connect UW-102 plug to the ribbon connector UW-18 number 1 of the main board
- 5 Fix the UW-102 mechanical surface mount
- 6 Insert the plug into the jack otherwise in the euroblock
- 7 Close the UW-102 connection module

